



SCREENING for COVID-19

Rapid results within 10-15 minutes !!

Rapid results within 10-15 minutes

Good for screening use

Simple operation without machine

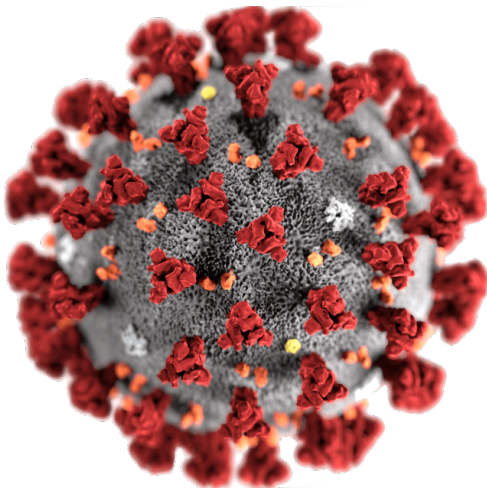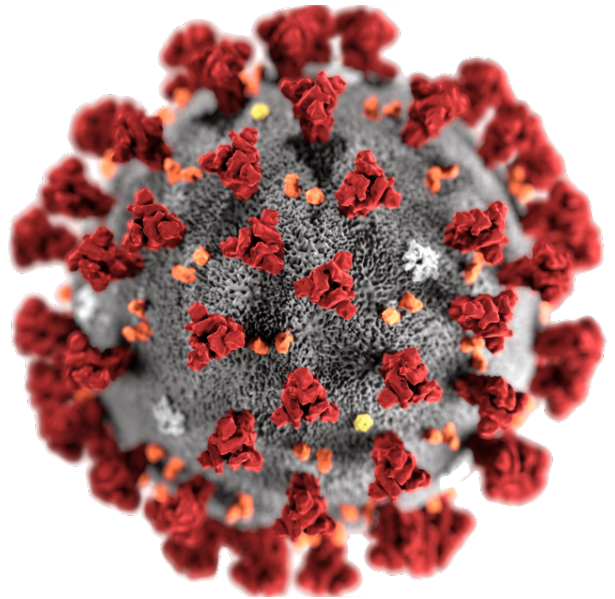

Serum, plasma & whole blood all applicable

Early detection to shorten the disease course

Specimen Requirement:

1. Suitable for human serum, plasma and whole blood samples. For plasma and whole blood, EDTA, sodium heparin, and sodium citrate anticoagulants are recommended.
2. The specimen should be kept or stored at 2-8 degree for no more than three days, otherwise store at -20 degree.
3. Do not use the specimen which are hemolyzed, cloudy or contain high lipids to avoid affecting the results.

Testing Method:

1. Recover the test specimen and reagents to room temperature.
2. Tear the outer package and take out test card.
3. Take 10ul specimen into the sample well, and immediately add 2 drops of diluents.
4. Let stand for 10 to 15 minutes to observe the results.



Result Interpretation:

